

## Folding Mount ROP (DFM2ROP)

The top horizontal tube of a Folding Mount can be lifted and rotated down to the side to move a device away. The mount is easily removed from the wheelchair and in the folded configuration forms a more compact unit for storage or transport. The Folding Mount ROP includes a locking style frame clamp receiver to secure for use on tilting seat frames. The mount is non-directional and may be installed on either side of a wheelchair.



### KEY FEATURES:

- Folds a device from in front down to the side of the wheelchair.
- Easily removed from wheelchair
- Suitable for most wheelchair types including Tilting Seat Systems

### DEFAULT CONFIGURATION:

- **Frame Clamp Inner Piece:**  
1" Standard UFC1000IP
- **Quick Release Base:**  
TUSB
- **Side tube 22" length, 3" offset:**  
S-22x3
- **Frame Clamp Assembly style:**  
Locking RFCR receiver

### CAUTION:

- In the folded position an attached device protrudes beyond the side of the wheelchair making it vulnerable to collision if the wheelchair is moved or driven. It is recommended to remove the device from the mount.
- When folded the unbalancing effect of heavy devices mounted to small, lightweight wheelchairs is reduced but not eliminated. It is advisable to remove the device and/or mount before the wheelchair is vacated.