

Positioner Mount (DPM9)

Achieve the ideal position and angle through 6 points of adjustment. The Positioner Mount allows simple and flexible positioning adjustment for height, angle and location through the use of two articulating joints and a height adjustment collar.



KEY FEATURES:

- Easy height adjustment
- 2 Articulating Joints for precise positioning
- Easily removed from wheelchair
- Suitable for Tilting Seat Systems

DEFAULT CONFIGURATION:

- **Frame Clamp Inner Piece:**
1" Standard UFC1000IP
- **Quick Release Base:**
TUSB
- **Frame Clamp Assembly Style:**
Locking RFCR receiver

CAUTION (DPM9):

- When installed on a tilting seat system the wheelchair seat should be in the upright position while adjustments are made. All joints must be tightly secured prior to tilting the seat and mount. (See Page 2)
- When adjusting and/or loosening any joint of the Positioner Mount the weight of the mount arm and attached device must be supported to prevent it from swinging or falling towards the user. Heavy or large devices should be removed during adjustments.

Mounts suitable for use on a tilting seat system or where an angled installation may be necessary:

- **Rigid Mount ROP Version (DRM1ROP)**
- **Mini Rigid Mount (M75RM)**
- **Mini Bent-tube Mount (M75BTM)**

The Removable Frame Clamp Receiver allows the mount to be locked onto the wheelchair and provide a secure attachment for use on a tilting seat system. Note the DRM1ROP, M75RM or M75BTM cannot be swung to the side; the mount must be lifted from the chair for access to the user and during transfers.

-
- **Folding Mount ROP Version (DFM2ROP)**
 - **Mini Folding Mount (M75FM)**

The Folding Mount ROP Version and Mini Folding Mount includes the ability to be locked at the frame clamp similar to the DRM1ROP.

-
- **Positioner Mount (DPM9)**
 - **Mini Adjustable Mount (M75AM)**

All adjustable joints must be tightened and secured before use, particularly on a tilt seat system. The frame clamp includes the locking feature to secure for use in a tilted or angled position. When adjusting or loosening any joint of the Positioner Mount or Mini Adjustable Mount the weight of the mount arm and attached device must be supported. Large or heavy devices should be removed during adjustment.

-
- **Swing Aside Mount (DSAM4)**

The Swing Aside Mount includes the Removable Outer Piece & Receiver components that allow the mount to be locked onto the wheelchair and provide a secure attachment for use in a tilted position.

Mounts not suitable for use on a tilting seat system or where an angled installation is necessary:

- **Locking Swing Away Mount (DLSA7)**

See [daessy.com](https://www.daessy.com) for more information on the DLSA7 & DLRFM8 mounts.

Mounts suitable for use on limited tilting seat system not beyond 15 degrees of vertical:

- **Rigid Mount (DRM1)**
 - **Folding Mount (DFM2)**
-